

Repose[®] Babytherm


Order Code: 6401100



Repose is an effective aid in the prevention of pressure ulceration for all patients including those assessed as very high risk, and for the treatment of existing pressure ulcers. Clinical supervision is advised if damage is severe.

To Inflate

- 1 Remove Repose from the tube, refit the two tubes together to form a pump
- 2 Connect the valve on the pump to the valve on the device - Push to fit
- 3 Inflate Repose until you hear air exhausting from the pump valve. Discontinue pumping and wait until you no longer hear air escaping
- 4 Cover the valve on the device using the solid end of the valve cap. Turn over with valve underside and away from patient. Secure with straps

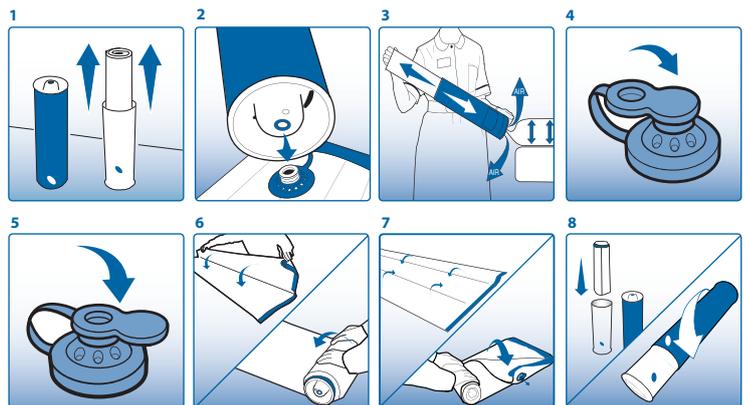

To Deflate

- 5 Insert the open end of the valve cap into the device valve. Air can now escape
- 6 Fold and roll up the device starting at the point furthest away from the valve until all air is expelled. Clean and allow to dry
- 7 Re-fold, and re-roll as tightly as possible
- 8 Replace device into the pump. Refit the two tubes together and rotate outer tube until it clicks closed

Please note, Repose is not suitable for the following:

- Persons weighing in excess of 139kg
- Persons with unstable spinal fractures
- Where body shape is such that the patient cannot be fully supported by Repose

Check device daily to ensure proper inflation level. In case of doubt, repeat inflation procedure with the user off the product. Avoid contact with heat and sharp objects

If deterioration in skin condition is noted, seek specialist advice.

REPOSE PRODUCTS CAN BE CLEANED AND REUSED



WIPE CLEAN



DO NOT DRY CLEAN



DO NOT TUMBLE DRY



DO NOT IRON

REPOSE PRODUCTS CAN BE CLEANED WITH SOAP OR DETERGENT AND WATER OR IN ACCORDANCE WITH LOCAL INFECTION CONTROL POLICY. RINSE WITH CLEAN WATER AFTER USING DISINFECTANT. ALLOW TO DRY BEFORE USE OR STORAGE. REPOSE IS SUPPLIED WITH A MANUFACTURERS WARRANTY OF 24 MONTHS FROM DATE OF DISPATCH.

IMPORTANT

CHECK PRODUCT DAILY TO ENSURE CORRECT INFLATION LEVEL. IN CASE OF DOUBT, REPEAT INFLATION PROCEDURES WITH THE PATIENT OFF THE PRODUCT. KEEP AWAY FROM SHARP OBJECTS AND DIRECT HEAT.