

User Instructions

USER INSTRUCTIONS

If the mattress is compact rolled, please refer to the section, '**Storage & Care.**' On delivery please open the mattress cover and place the inflatable Repose inner, clear side up inside the foam surround. Position the valve on the Repose in the corner highlighted on the outer cover.

Repose **ultracore** Mattress has been designed for use on solid / mesh / profiling / flatbed bases, for patients identified as being at high risk of developing a pressure ulcer, with a maximum weight of 222 Kg / 35 stones.

Ensure that the surface of the bed frame is clear from debris before placing the mattress – debris can damage the coating of the fabric.

The cover on this mattress is removable for cleaning purposes. Always ensure that, if removed, the cover is returned to the same inner. Using this cover on a different mattress core may render the warranty invalid.

The label on this mattress has a unique batch number which should be retained and used in the event of a warranty claim. NEVER remove the label.

Product Cover Marking

PRODUCT LABELLING



*65°C for 10 minutes or
73°C for 3 minutes



*bleaching agent
allowed



*Max user weight



*Do not pierce or cut



*Latex free



*Do not put near flame



Line dry



*do not iron



*CE conform



*Refer to user manual

REPOSE INNER MATTRESS PUMP UMBILICAL LOCATION



The Repose mattress umbilical is located inside the outer cover, bottom left hand corner as you stand at the foot of the bed. For inflating & checking of internal pressure.

